

## Foam Cervical Collar Wrap

The **ISO Preferred Foam Cervical Collar Wrap** provides healing support by reducing head and cervical vertebrae movement. The foam collar wrap is designed using medium density PU foam construction with a breathable polyester soft stockinette cover. The collar is lightweight for optimum comfort and support. A contour design under the chin supports the head in a neutral position, helps heal the neck after an injury and prevents further injury from occurring. The Foam Cervical Collar Wrap is flexible, easy to apply and adjusts for a comfortable fit.



ISO Preferred Foam Cervical Collar Wrap (NE37x) is intended to provide support for Cervical sprains and strains, Cervicalgia (pain in neck), Cervical Spondylotic Myelopathy (CSM), Whiplash injury caused by hyperextension and hyperflexion of cervical spine, Cervical Osteoarthritis, Neck pain and discomfort.

1. Loosen the closure strap at the back of the collar wrap.

1



2. Stand up straight and place the collar wrap around the neck with the closure strap at the back of the neck and the contour on the top front of the collar wrap just under the chin.

2



3. Wrap around neck. While holding the collar wrap in place with your left hand, lift the hook & loop fastening strap with your right hand and secure it in place so your neck is in a comfortable position.

3



4. If the fit is too tight or too loose, or the contour on the top front of the collar wrap is not in line under the chin, tighten and readjust as needed while ensuring the contour on the top front remains in line under the chin.

4



**DO NOT OVERTIGHTEN.**



## INTRODUCTION

Your **ISO Preferred Foam Cervical Collar Wrap (NE37x)** is designed to provide your body with the support and stability that it needs to successfully manage or eliminate your mobility concerns. It is critical that you follow your orthopedic professional's directions closely to ensure that the apparatus is worn properly and in the manner that will be most beneficial in addressing your issues. For correct application, please read the enclosed instructions and watch the on-line instructional video (if available) at [www.ISOpreferred.com](http://www.ISOpreferred.com) so that this device can provide you with superior long-term usability and care.

## USAGE RECOMMENDATIONS

It is vital that you read the included instructions and when applicable watch the instructional video for the **ISO Preferred Foam Cervical Collar Wrap (NE37x)**. The brace is intended for application by qualified professionals only. It is not recommended for patients to fit or apply or adjust their new brace without an orthopedic professional's expertise, unless otherwise instructed. Further questions regarding adjustments or changes, frequency of use, and/or duration of use should be presented to your orthopedic professional, as they are familiar with the specific needs and conditions, for which this device was prescribed.

## CARE AND CLEANING INSTRUCTIONS

Please do not place your brace in the washing machine or dryer at any time. When preparing to clean your **ISO Preferred Foam Cervical Collar Wrap (NE37x)** you will want to separate the parts where applicable and care for them individually.

- Do not dry clean, iron, bleach, or use an abrasive cleaning instrument on any part of the brace. Failure to follow these instructions may damage your brace and void the manufacturer's warranty and it may no longer work properly.
- Hand wash each section in cold water with a mild detergent. Lay flat to air dry.
- Allow all parts to air dry prior to reassembling the device.

## CAUTION

The **ISO Preferred Foam Cervical Collar Wrap (NE37x)** is for single patient use only. While every effort has been made to obtain maximum strength, function and durability, there is no guarantee that injury will be prevented through the use of this device, especially if the instructions are not followed correctly. **DO NOT OVERTIGHTEN.**